DISTRIBUTION CHAIN PARTICIPANT / IDENTIFICATION CHART

 PATRON Bohemia a.s. Českolipská 3419, Mělník, CZ (www.patron.eu)	Product identification label:
Manufacturer—product final inspection / name: Signature:	
Signature:	

Importer / EU-distributor : (name, address, contact info)	Importer / EU-distributor (stamp):
--	------------------------------------

Point of Sale / Service partner : (name, address, contact info)	Point of Sale / Service partner (stamp):
--	--

Date of sale (day, month, year): . . .	Name (block letters), phone number, stamp and signature of the seller:
---	--

INFORMATION FOR USE (en)

=> **medical device: rehab-stroller (buggy)**

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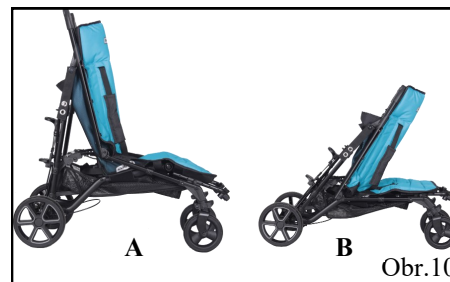
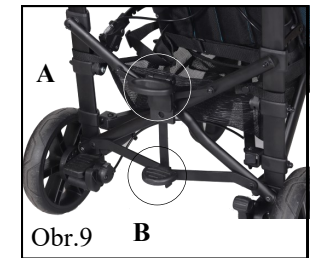
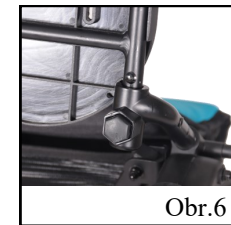
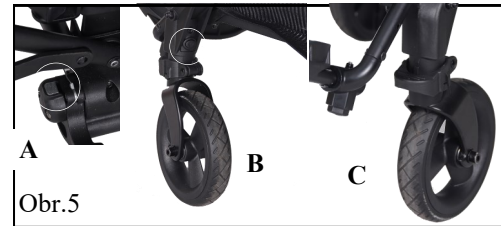
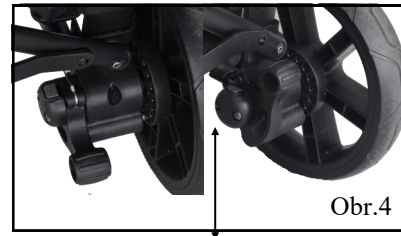
INTRODUCTION:

Dear customers, read this document carefully before using this product for the first time. Follow all safety instructions stated in this document. If you have any questions about the product and its proper use, contact professional staff or a service centre. Report any serious adverse event related to the product, or any deterioration in the user's health in connection with the use of the product, immediately to the official distributor of the product in your country or to the manufacturer of the product. For latest safety and/or recall information for this product, as well as the product declaration of conformity, please contact the official distributor of the product in your country or the product manufacturer. Keep this document in a safe place for further reference.

**..... WARNING, CAUTION:**

- ① **CAUTION! Any load attached to the handle (bag, ...) affects the stability of the buggy!**
- ② **Never leave your child unattended!**
- ③ **Recommended tire air-pressure: 180 kPa**

.....maximum load.

WARNING! Keep fingers and cover out of any moving parts when operating with the buggy.

WARNING! Never leave your child unattended! Even if the belts and brakes are in use.

WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.

WARNING! Check before use that seat unit is properly locked in the chassis interface.

WARNING! The seat unit is not suitable for a child younger 6 months.

WARNING! This product must not be used while jogging or in-line skating.

ATTENTION! Read this manual carefully before the first use of the buggy, as well as keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.

PRODUCT - DESCRIPTION, FEATURES, IDENTIFICATION:

You have received a product that is determined for versatile everyday usage outdoor, as well as indoor at home. Before using the product for the first time, get to know carefully how to use it, how to handle with it, how works all the product features, to avoid any possibility of injury. In particular, observe the following instructions:

- All users and / or their accompanying persons must be trained by qualified personnel how to operate with the product. Users and / or their accompanying persons are obliged to read and learn in particular about the residual risks of this product, in connection with the safety instructions given in this document.
- The product has been adapted to the individual needs of the user. Additional changes may only be made by qualified personnel.
- We recommend that you check the product customization once a year to ensure optimal product setup for users. Especially for users with changing anatomy (eg body dimensions and weight) it is recommended to adjust at least once every six months.

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- Your specific product may differ from the variants shown here. In particular, not all optional accessories described in this document are mounted on your product.
- The manufacturer reserves the right to make technical changes to the design described in this manual.

Product features:

This product is classified as a non-invasive therapeutic device of class 1. The product is intended for short-term and long-term use. The product is controlled by an accompanying person and is used to transport children with the help of another person. The product is intended for use on a solid surface in interiors and exteriors.

The product is used by children with temporary or permanent limitation of the ability to walk, unable to walk or keep standing. The product is suitable for users whose anatomy (eg body dimensions and weight) allows the use of this product for the given purpose and whose skin is intact. The product must only be used in conjunction with the accessories specified in the order form.

The manufacturer of this product assumes no responsibility for combinations with medical devices and / or accessories from other manufacturers. Only product combinations evaluated in terms of their efficiency and safety are permitted. This product and accessory combination must be permitted and stated in the product combination agreement.

Indications: mild to severe or complete restriction of the user's movement.

Contraindications: Absolute contraindications are not known. Relative contraindications - lack of physical or mental preconditions, significant surface damage of surface tissue (eg skin, etc.).

Accompanying phenomenon: Exceptionally, the following accompanying phenomena may occur when using the product - pain in the neck muscles, pain in the muscles and joints, circulatory disorders, risk of pressure sores. In case of these accompanying phenomena or problems, contact your doctor or therapist.

Product identification:

Information on the product label:

GTIN	Global trade identification number. According to GS1-standard = UDI-DI – Unique Device Identifier - Device Identifier. .
SN	Serial number. According to GS1-standard = UDI-PI – Unique Device Identifier - Production Identifier.
REF	Producer identification – product version
Mod.:	Product version / text
Spec.:	Product version / colour version
	Producer - address

Date of production: Serial number (first 3 digit) includes production date. („253“ = 253.-months from 01.01.2000 = January 2021)

SAFETY AND PROTECTION:

- The attendant must be familiar with the instructions of this manual.
- Professional medical advice regarding the suitability of this product must be sought before use.
- Always make sure, before departure, the child is comfortable, and that clothes and rugs etc, are tucked in, so they do not fall in the wheels.
- Ensure that child is clear of moving parts before making adjustments, to avoid any injury.
- When angle-positioning and adjusting the seat and backrest act accordingly to avoid sudden uncontrolled movements.
- The buggy MUST NOT be used as a seat for a child while traveling in motor vehicle or bus if the vehicle is not equipped for that and if the buggy is not crash tested model according to the requirements of ISO 7176-19 or additionally of ANSI/RESNA WC-4:2012 Section 19 Annex A.
- Do not allow the child to climb unassisted into and out from the buggy.
- Always apply the parking brakes, when getting in and out the buggy.
- The buggy is not designed to be driven down the stairs.
- Ensure yourself that the fixing accessories, safety bar and other equipment is fitted and adjusted correctly.

- Do not stand on the footrest.
- The brakes must be engaged at all times when the buggy is not moving.
- Getting over the curb practice firstly with an empty buggy.
- The attendant must take precautions against the fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the buggy.
- During the drive keep full attention, to avoid any dangerous situation or collisions.
- Do not overload the buggy. Overloading of a chassis, seat, bag or basket causes reduction of stability of the buggy.
- Make the full attention, when manipulate with the buggy, not to injure the child or yourself.
- If it is possible, always use drive-up ramps or lift to overcome any access restrictions.
- Always lift the buggy in two persons.
- Never transport the buggy when you and/or assistance are not sure if you can transport the buggy with the child safely.
- When you drive the buggy on the road, follow the road traffic regulations.
- Do not use the buggy for any other purposes than it is designed for.
- The buggy is designed to carry only one child.
- Do not use any other than original PATRON accessories designed for this product.
- When the buggy is folded and unfolded, the user guide must be followed carefully, to avoid any damage on the construction what could affect the product stability.
- In case of apparent defect, which limits product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product immediately and contact your local official distributor or service centre.
- Company PATRON does not bear any responsibility for the damages caused by using other than original PATRON accessories supplied by this company.
- Company PATRON does not bear any responsibility for the damages and accidents caused by inappropriate use of this product.

PREPARATION FOR USE, USE:

1) Assembling and adjusting the buggy:

Unfolding the buggy:

Put the folded buggy on the ground.

To assemble the front swivel wheels - take each swivel wheel and insert and push it into the wheel moulding until you hear a 'click' which indicates the wheel is locked in. Caution, necessary when first assembled.

To assemble the rear wheels - insert each wheel axle into the moulding and push in until locked. You should hear a 'click' which indicates the wheel has locked in. Caution, necessary when first assembled.

Release the transport safety-catch (pos. 18). Push the buggy partially into the upright position (Pic.10a). **Check the position of the 5-point belt plastic buckles to avoid that they cannot be jammed and subsequently broken during folding.** Stand between the rear wheels of the buggy and using your foot, push down the pedal (Pic. 9b) to unfold the buggy until you hear a 'click' which indicates the frame is fully opened and locked. Then secure the folding handle by pulling it up (Pic. 8) until the hinge handle clicks.

Fitting the footplate - Place the footplate into the bracket on both sides of the front frame and push until you hear a click. Tighten the footrest to make sure it is properly attached.

Folding the buggy:

Apply the parking brake.

To remove the footrest - Remove the footrest by pressing the buttons on both sides of the front frame (fig. 6).

Then release the backrest (fig. 7) and adjust the backrest to the upright position. Then release the guide by pressing the button on the guide joint and break it by pushing it downwards - always away from you / into the carriage (fig. 8). If the guide cannot be easily broken, make sure that the button is fully and correctly pressed and that the guide is folded in the right direction (according to the arrow)! Grasp the strut handle and pull upwards (Fig. 9A). Squeeze the buggy partially by pulling it towards you and place it forward (fig.10). Complete the assembly by turning on the transport lock (position 18).

To remove the front swivel wheels - Press the front wheel ejection button to remove the wheels (Pic. 5b).

To remove the rear wheels - Press the rear wheel release button to remove the wheels (Pic. 5a).

Upholstery:

Unpacking the buggy - The buggy is supplied with a partially removed cover. Before first use, it is necessary to attach the upholstery to the top of the chassis frame. Use 4 snap-fasteners to both sides of the chassis frame. Pressure on the cover to make sure it is properly fastened.

Removing and attaching the upholstery - From the top of the frame, release the cover with snap-fasteners. Next, loosen the Velcro fasteners and Release belts from buckles of the Five-point belt.

To attach the upholstery, follow steps in reverse order.

Seat upholstery is hand washable by lower temperature (max. 40°C).

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Footplate height adjustment:

Remove the locking pin of the footrest on both sides of the front frame (Pic. 2). Adjust the height of the footplate by sliding appropriately on both sides so that the thighs are almost parallel to the seat of the buggy. Do not lift the footrest too high, it could cause uncomfortable pressure on the buttocks. Then release the locking pin. Pull the footrest to ensure that it is properly fastened. You will hear a click.

Parking Brake:

To engage the brake, press brake lever at the rear wheel (Pic. 4), by pulling forward, Make sure the buggy is locked. To disengage the parking brake, lift up the brake lever.

Front wheel swivel lock:

If needed, the front wheel forks can be locked against vertical rotation.

The wheels are only locked in the direction of travel.

Press the lock lever down to lock the wheel. (Pic. 5c).

Make sure, with a slight pull on the wheels that they are both properly locked.

To unlock the wheels, set the lever to its upper position.

Adjustment of calf support:

Press the buttons on the outer sides of the support hinge (Pic. 3), set it to the desired angle and release the button.

Press gently on the support to make sure it is securely fastened. You will hear a click.

When lifting the calf support, the position is adjusted without pressing the buttons on the outer sides of the hinge.

Backrest recline:

It is possible to set 6 positions of the backrest (Pic. 1)

Grasp the backrest recline locking catch (Pic. 7) pull up the adjustment mechanism and tilt the backrest to the desired position. Release locking catch, press down on the backrest to make sure the backrest is secured (not positionable).

Standard equipment

Five-point belt:

Release all plastic quick fasteners, place the child into the buggy and adjust of desired lengths the belt. Fasten the belt and by pulling the belt, make sure it is secure.

Adjusting the length - Place the baby in the buggy, depending on the height of the child's shoulders, pull the shoulder straps through the opening of the cover (Fig. 11), pass the belts below the cover up to the opening in the backrest and push the shoulder strap through this hole (Fig. 12).

Basket:

Attach by snap-fasteners to the frame

Optional equipment:

Reductor

Raincover

Flexi-pads set – side pads for the seat width adjustment

Canopy Classic

Headrest adjustable

Footmuff-Summer

This product can be optionally supplied with very wide range of fixation and positional accessory. Please find the optional accessory user guide as separate brochure.

2) buggy in operation:

Getting into the Buggy:

ensure the parking brakes are on, to make access easier, the footrest can be taken off; if possible, place the back of the buggy against a wall or immovable object (alternatively, another helper may hold the buggy steady); place child carefully into the buggy, fasten safety belt and fit the footrest. The child must always feel comfortable.

Getting out of the buggy:

Engage parking brake. If necessary ask trustful person for help to hold the buggy firmly. If required to easy getting out of the buggy adjust footrest position Take off all applied fixation and positioning accessories. Take off the safety bar and get the child carefully out of the buggy.

Getting over the curb stones and low obstacles:

Down - Drive the front wheels to the edge of the curb. Hold the handles firmly and tip the buggy backwards and balance on the rear wheels. Drive the rear wheels to the edge of the curb, lower the buggy down the curb slowly on its rear wheels.

Up - Drive the front wheels close to the curb. NEVER bump the front wheels into the curb stone!. Hold the handles firmly and tip the buggy backwards and balance on the rear wheels. Drive forward and place the front wheels on top of the kerb. Position the rear wheels at the curb tone bottom edge and lift to the upper level.

Manual transport of the buggy:

- make use of ramps and lifts when they are available
- if there is no alternative and steps should not be avoided at least one trustful assisting person must be available to give assistance
- do not attempt to take the buggy up or down over the number of steps unless you and your assisting person are not sure to have the full control during the manipulation as well as he or she can absorb the weight of its occupant and buggy.
- take the rests in regular intervals, if you are tired
- occupant hands must be inside the seat unit during the manual transport time
- NEVER lift the buggy holding the safety bar or footrest!

Going stairs downwards:

Tightly hold the handle and position the buggy squarely with the front wheels on the top step. Tip the handle backward and push the buggy forward until the rear wheels are at the end of top step. Assisting person stands in front, holding the buggy by gripping the front frame. Lift the buggy by both persons at once. Carry the buggy down the steps. Continue step by step to the bottom surface.

Going stairs upwards:

Tightly hold the handle and position the buggy squarely with the rear wheels against the bottom step. Tip the handle backward and push the buggy forward until the rear wheels are at the end of bottom step. Assisting person stands in front of buggy, holding the buggy by gripping the front frame. Lift the buggy by both persons at once. Carry the buggy upstairs. Continue step by step to the top surface.

Carrying a folded buggy :

Always carry the buggy with secured fixation strap. Hold on the main frame only. Holding on other parts not allowed!

MAINTENANCE, PRODUCT DISPOSAL:

- your buggy has been designed to a high standard and with correct use and maintenance will give many years of trouble free service
- regularly inspect locking devices, catches, footrest, joints fixings to ensure they are secure and in full working order
- brakes, wheels, tires are subjects to wear and should be replaced if necessary by a repairer approved by the buggy supplier
- we recommend that the buggy is checked by an approved repairer every 12 month.
- we recommend that after 24 months visit a service department.
- if the buggy is damaged or in need of repair, the approved repairer will be able to advise on what is required.
- alternatively, PATRON Bohemia a.s. Customer department is available for product information.
- only PATRON Bohemia a.s. replacement parts should be used on this buggy.
- should parts of your buggy become stiff or difficult to operate (or not later than every 6 month), ensure that dirt is not the cause, apply a light application of a spray lubricant e. g. WD 40 (do not use oil or grease). Ensure that the lubricant penetrates between the components but do not over spray as this may encourage the accumulation of dirt. Wipe off any excess spray. Regular lubrication of moving parts will extend the life of your buggy and take opening and folding easier.
- make sure that you are familiar with the operation of the buggy. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions.
- wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.
- the seat fabric can be washed by hands (max. 30°C).
- never clean with abrasive, ammonia based, bleach based or spirit type cleaners.
- after exposure to damp conditions, dry off with a soft cloth and leave fully open in warm environment.
- store your buggy in a dry, safe place.
- Check and tighten the screw connections regularly.
- If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.

Product lifetime: The expected product lifetime is 5 years. The expected product lifetime was used as a basis for product development and the definition of requirements for the use of the product for the given purpose. This data also includes maintenance requirements to ensure the required efficiency and safety of the product.

Disposal instructions: For disposal, the product must be disposed of in accordance with the local valid environmental protection regulations in the country, where the product was used.

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WARRANTY CONDITIONS:

1. On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
2. The product has to be used entirely for the purpose it is designed for and the instructions in the user guide must be followed.
3. The product has to be properly used, maintained, cleaned and stored in accordance to the user guide instructions.
4. Do not pull down the identification tag. It has to be readable and undamaged for the whole warranty period.
5. The warranty case has to be claimed by the seller, where the product was bought. Properly filled "Warranty sheet" and the proof of purchase must be declared by the warranty application. Every defect should be claimed immediately
6. The claim procedure can be started when the device is delivered complete, in original packaging, carefully packed and secured against damage. The owner is committed to deliver the device clean in accordance to the general hygienic measures. The basic maintenance and cleaning cannot be expected or even required when claimed.
7. The warranty does not cover: failures caused by the overloading of the products, accident, forced damage, unauthorized changes made on the product and/or its accessories, natural wear and tear of parts, especially upholstery dirt and damage, wear of plastic parts, brakes, tyre profile and tyre puncture, etc.
8. The warranty is terminated whenever the following conditions arise: the user fails to follow the instruction mentioned in this user guide and manufacturer's storage and maintenance suggested procedures, or whenever the product is serviced by any other party except of authorised service centres, or when the identification label is not sticked on the buggy and also in the warranty sheet.
9. The warranty period cannot be interchanged with using period, that means for the lifetime period when correct use and maintenance of the product in accordance to the user guide are fulfilled.
10. The producer nor the importer is not liable for any loses and injuries caused by manipulation and usage of the product in conflict with the "User Guide" instructions.

The manufacturer, PATRON Bohemia a.s., reserves the right to make changes to the products.

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SCHEMA DESCRIPTION

- | | | | |
|-----|--------------------------------|-----|------------------------|
| 1. | handle | 18. | transport safety-catch |
| 2. | folding hinge handle | 19. | backrest |
| 3. | safety belt | | |
| 4. | calf rest | | |
| 5. | hinge of calf support | | |
| 6. | footrest | | |
| 7. | locking pin of the footrest | | |
| 8. | buttons to remove the footrest | | |
| 9. | front wheels | | |
| 10. | swivel lock | | |
| 11. | front swivel castor wheels | | |
| 12. | rear wheels | | |
| 13. | buttons to remove rear wheels | | |
| 14. | brake lever | | |
| 15. | pedal to unfold | | |
| 16. | grip to fold | | |
| 17. | backrest recline locking catch | | |

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