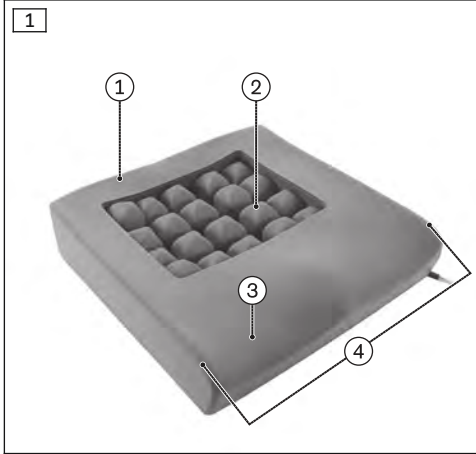
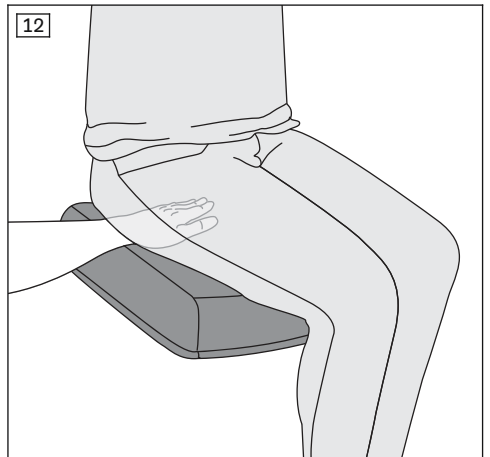
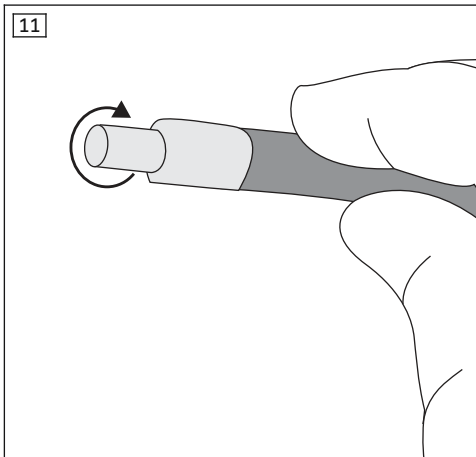
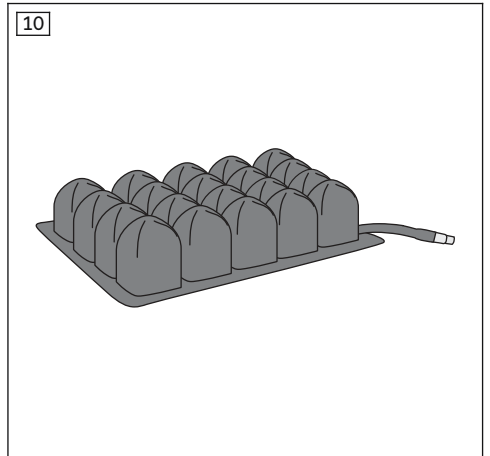
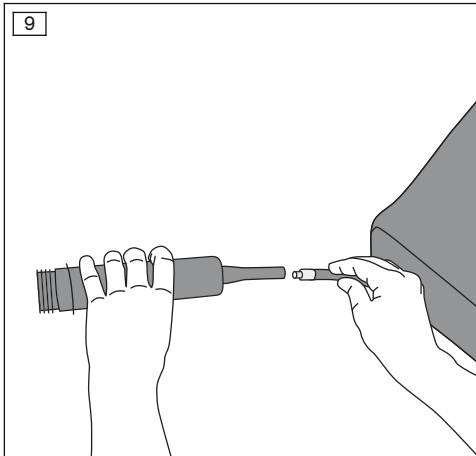
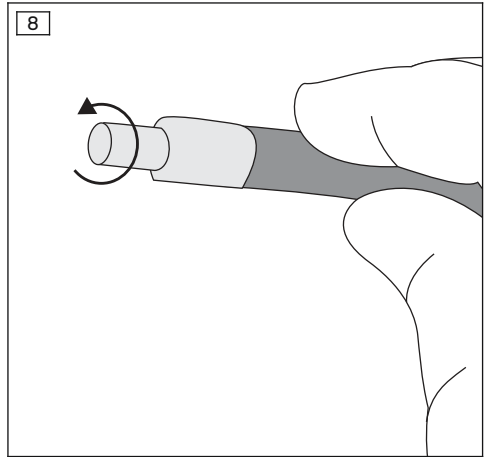
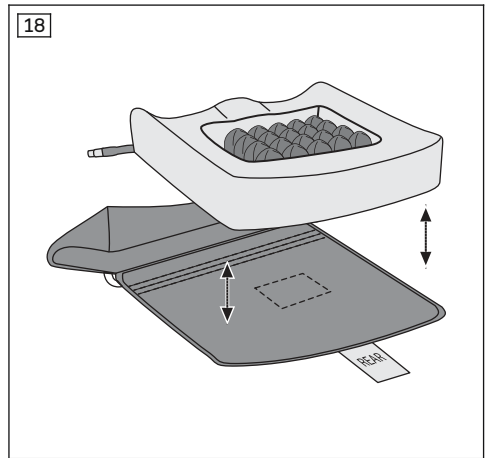
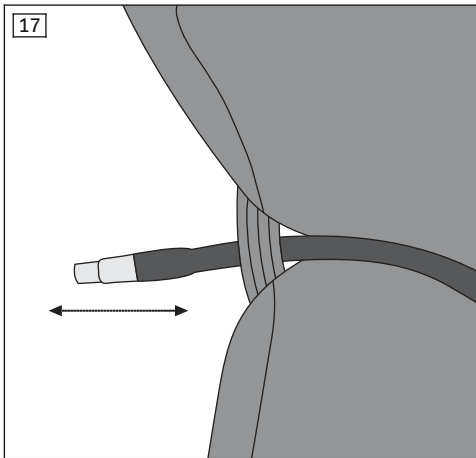
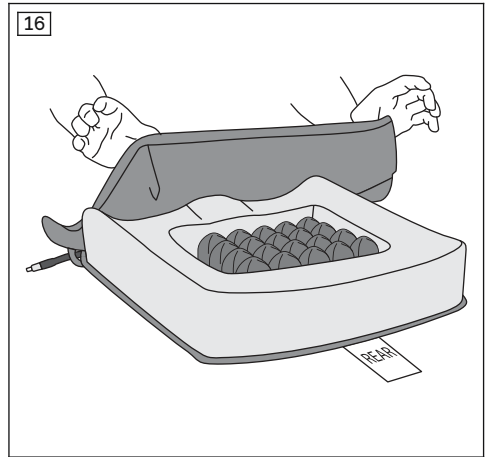
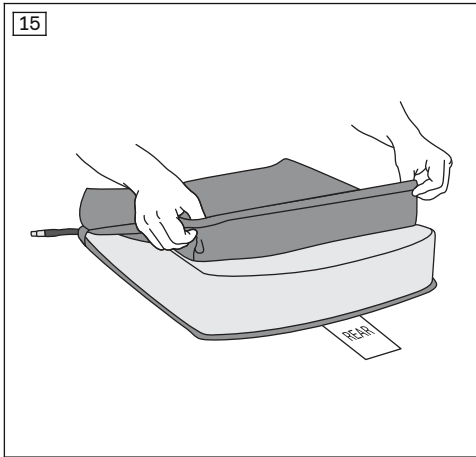
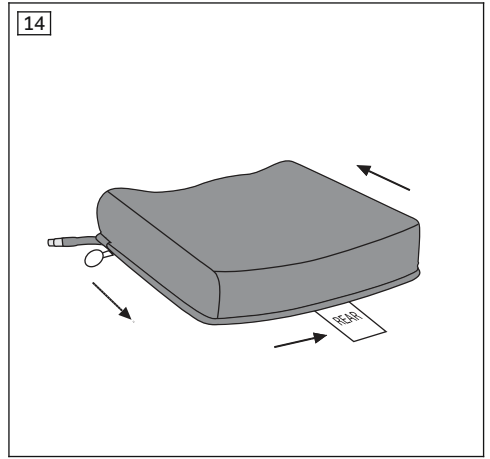
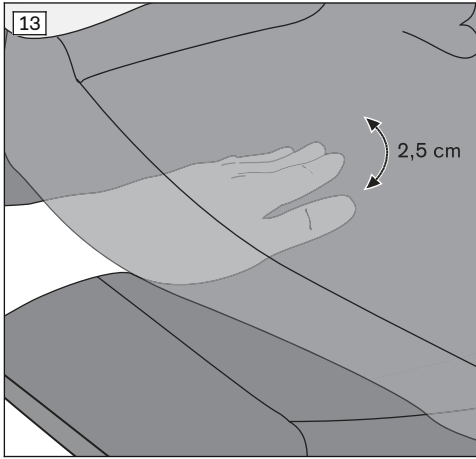
Terra Flair

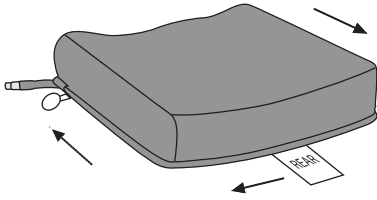
DE	Gebrauchsanweisung	6
EN	Instructions for use	17
FR	Instructions d'utilisation	27
IT	Istruzioni per l'uso	39
ES	Instrucciones de uso	50
PT	Manual de utilização	61
NL	Gebruiksaanwijzing	71
SV	Bruksanvisning	81
DA	Brugsanvisning	91
NO	Bruksanvisning	101
FI	Käyttöohje	111
PL	Instrukcja użytkowania	121
HU	Használati utasítás	132
CS	Návod k použití	142
HR	Upute za uporabu	152
SL	Navodila za uporabo	162
SK	Návod na používanie	172
TR	Kullanma talimatı	182
EL	Οδηγίες χρήσης	192
RU	Руководство по применению	204
JA	取扱説明書	216
ZH	使用说明书	226
ZH*	使用說明書	235
KO	사용 설명서	244

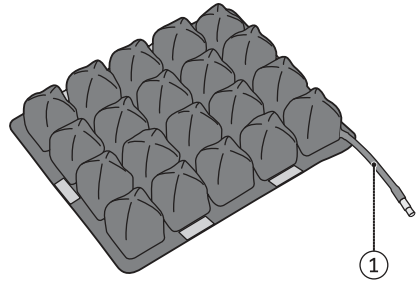
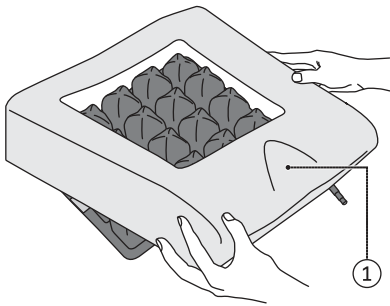
19

20

21

INFORMATION

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INFORMATION

Date of last update: 2020-04-15

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

INFORMATION

- ▶ New information regarding product safety and product recalls as well as the declaration of conformity can be obtained at oa@ottobock.com or from the manufacturer's service department (see inside or outside of back cover for addresses).
- ▶ You can request this document as a PDF file at oa@ottobock.com or from the manufacturer's service department (see inside or outside of back cover for addresses). The PDF file can also be displayed in a larger size.

These instructions for use provide important information regarding the installation and proper use of the 476C00=SK87*/88* Terra Flair wheelchair seat cushion (custom size: 476C00=SA880).

All users and/or their attendants must be trained by qualified personnel in the use of the product with the aid of these instructions for use. In particular, users and/or attendants must be informed of the residual risks with the aid of the safety notices.

Please note the following:

- It may be necessary to adapt the product according to the needs of the user and the medical conditions. Adjustments may only be made by qualified personnel and in close coordination with the user.
- The manufacturer reserves the right to make technical changes to the model described in these instructions for use.

2 Product description

The wheelchair seat cushion is used as a decubitus prophylaxis. It provides optimal pressure redistribution and reduces shear forces.

The seating surface consists of an anatomically shaped rest suspension foam base (see fig. 1, item 1) with an integrated air insert in the rear section (see fig. 1, item 2). The foam base distributes pressure over the largest possible surface area, thereby reducing pressure points. The seat cushion provides postural stability due to the anatomically shaped recesses for positioning the thighs and the ischium bone (see fig. 1, items 3, 2). The air insert can be individually adapted by means of air pressure. This provides ideal protection for the most sensitive bony areas.

The foam base and air insert form the core of the seat cushion (see fig. 1) and are protected by a liquid-repellent cover (see fig. 2).

The components of the wheelchair seat cushion have the following functions:

Component	Figure	Function
Rest suspension foam base, anatomically shaped	see fig. 3	Lateral and medial thigh support, posture stability through secure positioning of the legs
High air insert with dry floatation ¹⁾	see fig. 4	Encloses and envelopes most sensitive bony areas more effectively for optimal pressure redistribution under these areas
Cover made of PU-coated fabric, stretchable in two directions, liquid-repellent	see fig. 2	Suitable for incontinence

¹⁾ The dry air floatation principle enables weight to be distributed evenly without shear forces.

3 Intended use

The safe use of the product can be ensured only if it is used as intended in accordance with the information contained in these instructions for use.

3.1 Indications for use

The product is intended **exclusively** as a wheelchair seat cushion for adults with intact skin who use a wheelchair on a temporary or permanent basis.

The optimally suited seat cushion must be selected by trained, qualified personnel depending on the size and individual sitting needs of the user. It may only be used with a wheelchair in the appropriate size.

The allowed user weight is **35 kg [77 lbs] to 150 kg [330 lbs]**. We assume no liability if the product is used outside the allowed user weight range.

3.2 Indications

- Minor to pronounced or complete restrictions of mobility
- Increased decubitus risk

3.3 Contraindications

3.3.1 Absolute Contraindications

- None known

3.3.2 Relative Contraindications

- Failure to meet physical or mental requirements

3.4 Further Usage Restrictions

The product is designed for use by **one person** only. Re-use of the product is not permitted due to hygiene considerations.

3.5 Qualification

This product may only be selected and adapted by qualified personnel. Compliance with all manufacturer specifications and all applicable legal provisions is required. Please contact the manufacturer's service department for further information (see inside or outside of rear cover for addresses).

4 Safety

4.1 Explanation of warning symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
 CAUTION	Warning regarding possible risks of accident or injury.

NOTICE

Warning regarding possible technical damage.

4.2 General safety instructions

⚠ WARNING**Improper handling of packaging materials**

Risk of suffocation due to neglect of the duty to supervise

- ▶ Packaging materials must be kept out of the reach of children.

⚠ CAUTION**Altered centre of gravity resulting from use of the seat cushion**

Risk of tipping over or falling due to unstable settings

- ▶ After installing the seat cushion in the wheelchair, always check the user's sitting position and the wheelchair's centre of gravity. The sitting position and centre of gravity must be appropriate for the intended use and the user's requirements. Work with the user to find the optimal sitting position.
- ▶ If necessary, adjust the position of the seat and axle to improve stability. Alternatively assess the suitability of anti-tip devices.

⚠ CAUTION**Lack of testing prior to long-term use**

Risk of skin reddening or pressure sore formation due to incorrectly selected seat cushion

- ▶ Any seat cushion should be tried out by the user under medical supervision for several hours prior to long-term use. The skin should be monitored for redness and/or other damage.

NOTICE**Use of an incorrect air pump**

Damage to the product

- ▶ Only use the supplied hand air pump for inflating the air insert.

4.3 Safety instructions for use

⚠ WARNING**Improper handling of ignition sources**

Risk of severe burns due to seat cushions, padding and upholstery catching on fire

- ▶ The seat cushions, padding and covers fulfil the normative requirements for flame resistance. However, they may still ignite if fire is handled improperly or negligently.
- ▶ Keep away from all ignition sources, especially lit cigarettes and open flames.

⚠ WARNING**Neglecting the duty to supervise**

Risk of suffocation due to small loose parts

- ▶ Note that the product includes small parts that can be loosened and removed without tools.
- ▶ Ensure that small children, for example, do not swallow them.

⚠ WARNING**Exceeding the service life**

Serious injuries due to failure to observe the manufacturer's requirements

- ▶ Using the product beyond the specified expected service life leads to increased residual risk.
- ▶ Observe the specified service life.

⚠ WARNING

Independent modification of settings

Serious injuries to the user due to improper changes to the product

- ▶ Do not modify the settings established by the qualified personnel. Only the settings described in the section "Use" in these instructions for use may be adjusted independently.
- ▶ In case of problems with the settings, please contact the qualified personnel who adjusted your product.

⚠ CAUTION

Skin damage

Skin damage or pressure points due to overloading

- ▶ Check your skin for intactness before and during use of the product.
- ▶ Pay attention to diligent skin care and pressure redistribution during interruptions in using the product.
- ▶ If skin damage or other problems occur during use, stop using the product. Consult the qualified personnel.

⚠ CAUTION

Failure to observe care instructions

Risk of pressure sores due to incorrect skin care

- ▶ All seat cushions were developed by the manufacturer to reduce pressure on the buttocks. A seat cushion alone, however, cannot eliminate sitting pressure or possible pressure sores entirely.
- ▶ When using the seat cushion, always pay attention to correct skin care – especially through a healthy diet, hygiene and regular pressure relief.

⚠ CAUTION

Extreme temperatures

Hypothermia or burns due to contact with components, failure of components

- ▶ Do not expose the product to any extreme temperatures (e.g. direct sunlight, sauna, extreme cold).
- ▶ Do not leave the product in the immediate vicinity of heaters.

⚠ CAUTION

Use of the product during diagnostic examinations and therapeutic treatment

Impairment of the examination results or the effectiveness of treatment due to interactions of the product with devices that are used

- ▶ Make sure that examinations and treatments are carried out exclusively under the prescribed conditions.

NOTICE

Improper use

Risk of damage to the cushion surface due to user error

- ▶ Do not allow the wheelchair seat cushion to come into contact with sharp objects. This also applies to animals with sharp claws such as pet cats.

- ▶ Do not expose the wheelchair seat cushion to excessive heat or open flame.
- ▶ If the wheelchair seat cushion is expected to come into contact with liquid, such as spilt drinks or episodes of incontinence, always use it in conjunction with a liquid-repellent cover.
- ▶ Only use the official Ottobock incontinence covers for the product. The incontinence covers have been optimised for the wheelchair seat cushion to maintain the pressure redistribution capability of the seat cushion. Contact your prescribing therapist, clinician or specialist dealer to obtain a spare Ottobock cover.

NOTICE

Wear of seat cushions, padding and upholstery

Loss of functionality due to prohibited continued use

- ▶ Have the seat cushions, padding and upholstery replaced promptly in case of damage.

4.4 Nameplate

Label	Meaning
	A Additional information (optional)
	B Manufacturer's logo
	C Manufacturer's reference number
	D Manufacturer's product name
	E Global Trade Item Number (DI) ¹⁾
	F Textile care instructions with symbols according to ISO 3758
	G Textile labelling
	H Manufacturer information/address
	I Symbol for medical device
	J WARNING! Read the instructions for use before using the product. Observe important safety-related information (e.g. warnings, precautions).
	K CE marking
	L Manufacturing date ²⁾
	M LOT or serial number (PI) ³⁾ incl. barcode
	N Seam
	O Sewn in, not visible: LOT or serial number Manufacturer's reference number

¹⁾ UDI-DI to GS1 standard; UDI = Unique Device Identifier, DI = Device Identifier

²⁾ YYYY = year of manufacture; MM = month of manufacture; DD = day of manufacture

³⁾ UDI-PI to GS1 standard; UDI = Unique Device Identifier, PI = Product Identifier

4.5 Side effects

The following side effects may occur during use of the product:

- Circulatory disorders, risk of pressure sores

Contact a doctor or therapist in case of problems.

5 Delivery

5.1 Scope of delivery

INFORMATION

- ▶ To order spare parts, please contact the qualified personnel.
- ▶ When ordering spare parts, please specify the width and depth of your product. Check the sewn-in label on the cover for information on the dimensions.

The product is delivered preinstalled. The scope of delivery includes:

- Seat cushion (components see Page 17)
- Accessories: hand air pump (see fig. 6), repair kit for air insert
- Instructions for use

5.2 Storage

Store the product in a dry, enclosed room with sufficient air circulation and protection from external influences. Specific information about storage conditions: see Page 27.

6 Preparing the product for use

INFORMATION

- ▶ The procedures described in this section may only be carried out by qualified personnel.
- ▶ These instructions for use must be given to the user along with the product.

INFORMATION

- ▶ The seat cushion must be set up by qualified personnel before delivery.
- ▶ Before handing over to the user or attendant, perform sufficient seat tests under medical supervision. Have the seat cushion tested by the user as needed.

6.1 Setting Up

- 1) Place the wheelchair seat cushion on the wheelchair, making sure it is centred and with the contoured side up. The valve must be located at the front left (see fig. 7).
- 2) Turn the valve anti-clockwise to open it (see fig. 8).
- 3) Attach the flexible tube of the pump to the valve (see fig. 9).
- 4) Inflate the seat cushion until it begins to arch upwards slightly (see fig. 10).
- 5) Close the air valve by twisting it clockwise and disconnect the pump from the valve (see fig. 11).
- 6) Lower the user down carefully onto the wheelchair seat cushion. The seat cushion must be centred below the user.

INFORMATION: Ensure the individual is sitting in the desired sitting position.

- 7) Slide your hand palm down between the cushion surface and the user's buttocks (see fig. 12).

INFORMATION: You should be able to feel the ischial tuberosity.

- 8) Keep your hand under the user's ischial tuberosity and rotate the valve anti-clockwise to slowly let out air (see fig. 8).
- 9) Allow the air insert to sink until your fingertips can only move up and down over a distance of **20 mm – 25 mm** (see fig. 13).
- 10) Close the valve by turning it clockwise (see fig. 11).

- 11) Wiggle your fingers to ensure the correct clearance. If there is **less than 20 mm** clearance between the user and the wheelchair seat, air must be added and the process repeated.

INFORMATION: A hand check should be performed daily for all dry air flotation cushions. The seat cushion needs to be readjusted only when there is a change in clearance during a hand check, or after a puncture in the air insert has been repaired.

- 12) Slide the valve into the cover and close the zip.

7 Use

7.1 Positioning

⚠ CAUTION

Damage to the product, unexpected noise or odours

Health problems for the user due to improper handling

- ▶ Stop using the product immediately in the event of defects or other hazards that can result in personal injury, or if there are noises or smells that greatly differ from the delivered condition of the product.
- ▶ Contact the qualified personnel.
- ▶ Report damage to the product that may cause health problems for the user to Ottobock.

INFORMATION

- ▶ Place the seat cushion in the middle of the seat so that there is sufficient space on all sides for the wheels and frame.
- ▶ Optimum pressure redistribution is ensured when the leg support of the wheelchair is adjusted so that the thigh of the user completely rests on the seat cushion.
- ▶ Regularly remove dust and dirt particles from the hook-and-loop strap at the bottom so that the seat cushion remains firmly in place.

Correct positioning of the seat cushion in the wheelchair must be maintained for daily use:

- The label that says "REAR" points towards the back support (see fig. 5).
- The flat side points downwards, the contoured side points upwards.
- The hook-and-loop strap on the cover faces the wheelchair's seat bottom. It is used to attach the seat cushion in the wheelchair.

7.2 Removing/putting on the cover

Removing the cover

The cover can be removed for washing the seat cushion:

- 1) Open the zip on all three sides of the cover (see fig. 14).
- 2) Lift up the cover from the seating surface of the cushion (see fig. 15; see fig. 16).
- 3) Pull the tube on the air insert out of the cover (see fig. 17).
- 4) Carefully remove the foam base with the air insert from the cover (see fig. 18).

INFORMATION: Cleaning information see Page 25.

Putting on the cover

- 1) Ensure the air insert is in the correct position (see Page 23).
- 2) Open the zip on all three sides of the cover (see fig. 14).
- 3) Put the seat cushion into the cover with the zip running along the rear and side edges (see fig. 18).
- 4) Push the tube on the air insert into the cover (see fig. 17).
- 5) Pull the cover over the cushion (see fig. 16; see fig. 15).
- 6) Zip up the cover again (see fig. 19).
- 7) Ensure the cover fits well and there are no large folds or wrinkles on the top surface.

7.3 Cleaning

NOTICE

Improper cleaning of the wheelchair seat cushion

Risk of damage to seat cushion due to user error

- ▶ Do not machine wash the foam components.
- ▶ Do not dry the seat cushion and its components in the tumble dryer.
- ▶ Gently wipe the surface of the foam components with a soft, damp cloth or sponge; do not rub. Do not use a brush or similar on the surface.
- ▶ To prevent mould formation, the foam components must be completely dry before putting them into the cover.
- ▶ Do not wash the cover at a temperature higher than specified. The cover can shrink when washed at higher temperatures. This could impede the seat cushion's ability to redistribute pressure and damage the fabric.
- ▶ Do not use any bleaching agents or fabric softeners for cleaning.

INFORMATION

- ▶ In case of advanced wear and tear, have the components that are available as spare parts replaced by qualified personnel.
- ▶ To disinfect the cover, add a standard hygiene detergent during washing.
- ▶ Use an additional incontinence cover in case of heavy incontinence. Observe the care and cleaning instructions for the incontinence cover.

Clean the product every month.

7.3.1 Cleaning the cover

- 1) Unzip and remove the cover for separate washing (see Page 24).
- 2) Close the zipper on the cover before washing.
- 3) The cover can be washed at +60 °C [+140 °F] using a mild, environmentally friendly detergent.
Recommendation: Wash the cover using the delicate cycle at **+40 °C [+104 °F]** to prevent excessive wear and tear.
- 4) Allow to air dry.

7.3.2 Cleaning the air insert and foam base

- 1) Remove the air insert and foam base from the cover for cleaning (see Page 24).
- 2) Use lukewarm water for cleaning. If necessary, an alcohol-free disinfectant or mild detergent can be added.
- 3) Wipe the air insert and foam base with a damp cloth or sponge.
INFORMATION: When adding disinfectant or detergent, wipe the air insert and foam base with clear water in a second step.
- 4) Dry the air insert and foam base with a towel, dab or let air dry at room temperature.
- 5) Avoid exposure to direct heat (e.g. sun or heat from radiators and stoves).

8 Maintenance and repair

8.1 Maintenance

- The function of the product should be checked **before each use**.
- Check the product **monthly** for wear and tear and damage. The interval between inspections should be further reduced for very heavy users.
 - Check the cover for rips, holes and other damage.
 - Verify that the zip is working properly.
 - Remove dust and dirt particles from the cover's hook-and-loop strap.

- Check foam base and air insert for damage.
- Have damaged or worn parts replaced by qualified personnel as soon as possible.
- Servicing and repairs (except on the air insert, see Page 26) may only be carried out by qualified personnel or the manufacturer. Only Ottobock spare parts are used for repairs.

8.2 Repair

INFORMATION

- ▶ Do not continue using the seat cushion if the air insert is leaking due to damage.
- ▶ If it is not possible to repair the air insert yourself, contact the qualified personnel.

Removing the air insert

- 1) Remove the cover (see Page 24).
- 2) Gently ease the air insert and valve away from the foam. Do not pull on the air cells.

Sealing a puncture

- 1) Locate the puncture.

INFORMATION: If the puncture is not visible, inflate the air insert. Close the valve and immerse the air insert in water. You can locate the puncture in the air insert by the bubbles.

- 2) Follow the repair instructions provided with the repair kit.
- 3) Reinflate the air insert.
- 4) Immerse the air insert in water. Make sure the puncture has been sealed.

Inserting the air insert

- 1) Dry the air insert thoroughly.
- 2) Lay the air insert on a table. The tube faces forward (see fig. 20, item 1).
- 3) Position the foam base over the air insert. The wedge faces forward (see fig. 21, item 1).
- 4) Fasten the foam base to the air insert using the hook-and-loop closures so it is secured against sliding.
- 5) Put on the cover (see Page 24).
- 6) Set up the seat cushion for the user (see Page 23).

9 Disposal information

Return the product to the qualified personnel for disposal.

All components of the product must be disposed of properly in accordance with the respective national environmental regulations.

10 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

10.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

10.2 Warranty

Further information on the warranty terms and conditions can be obtained from the qualified personnel that has fitted this product or the manufacturer's service (see inside back cover for addresses).

10.3 Service life

Expected lifetime: **2 years**

The design, manufacturing and requirements for the intended use of the product are based on the expected lifetime. These also include the requirements for maintenance, ensuring effectiveness and the safety of the product.

11 Technical data

Dimensions, weights, temperatures	
Dimensions (depth x width)	See label on the cover
Weight ¹⁾ [kg (lbs)]	1.4 (3)
User weight [kg (lbs)]	35 (77) – 150 (330)
Functional heights ¹⁾	
Ischial zone [mm (")]	Adaptable (see fig. 1, item 2)
Support for trochanters [mm (")]	75 (2.9)
Thigh trough [mm (")]	60 (2.3) (see fig. 1, item 3)
Edge zone [mm (")]	95 (3.7) (see fig. 1, item 4)

Component	Material
Foam base	Rest suspension PU foam
Foam density (ISO 845) [kg/m ³ (lbs/ft ³)]	100 (6.24)
Foam firmness (ISO 3386) [kPa]	11
Air insert	Neoprene rubber
Cover	PU-coated fabric, washable

¹⁾ Reference size 405 x 405 mm (16" x 16"), in ready-to-use condition, without incontinence cover

Ambient conditions

Temperatures and relative humidity	
Temperature during use [°C (°F)]	0 to +40 (+32 to +104)
Transport and storage temperature [°C (°F)]	0 to +40 (+32 to +104)
Relative humidity [%]	10 to 85; non-condensing

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India Pvt. Ltd.
Otto Bock Korea HealthCare Inc.
South East Asia Co., Ltd.
Otto Bock Mobility Solutions GmbH